



Element RC Update Installation Guide

An empty USB drive is required.

1. Download software update file 1.00.01.44.zip
<https://www.scilvet.com/downloads/>.
2. Create new folder named "upgradepackage" on your USB stick.
3. Copy downloaded ZIP-file "1.00.01.44" in this folder "upgradepackage"
4. Turn off the instrument.
5. Insert the USB drive into the lower USB port of the Element RC
6. Turn on the instrument.
7. Follow the instructions on the screen and confirm with OK.
8. Turn the Element RC off after the update is installed and the message is displayed on the screen.
9. Remove the USB drive.
10. Turn on the instrument: the update is completed.
11. Check the update version on the main screen of the instrument.