


How To Remove A Mixing Chamber Blockage

Vet abc Technical Help

1. Open front door of the Vet abc.
2. If there is a black cover over your mixing and RBC chamber you must remove it by loosening the top and bottom screws and sliding the cover upwards, then pulling it off.

Screw # 1

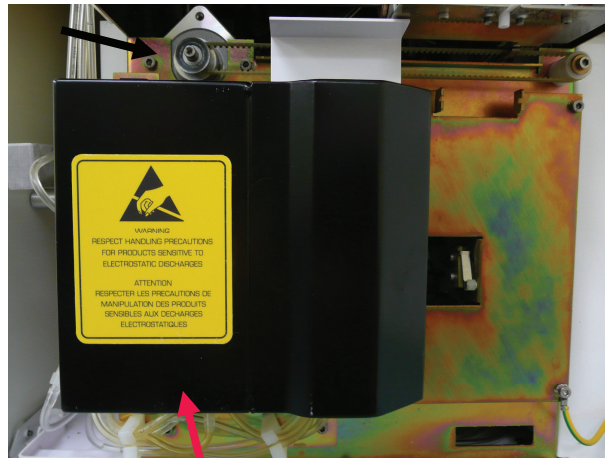

Screw # 2 is located inside the black box. You will need to put the vet abc on the edge of a table and bend down to see it.

3. Pull the mixing chamber out of the clip.




How To Remove A Mixing Chamber Blockage

Vet abc Technical Help



4. Remove the tubing from the T

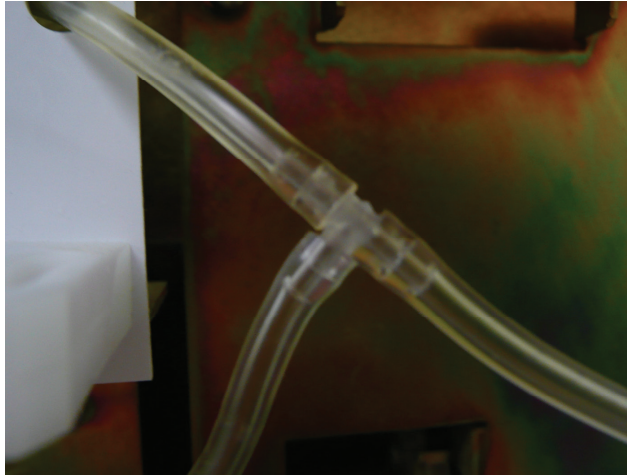

5. Inspect the T for clots.
6. If you see a clot suck it out using a syringe. Flush out the T using a syringe with bleach
7. Reattach the tubing as it was originally.
8. Push the mixing chamber back into the clip.
9. Put the black cover back on. Be sure not to pinch any tubing.
10. Run one concentrated cleaning.
11. Run a startup cycle.
12. Run a blood control. If your controls are not in range fax your blood control results to scil animal care company at 847-223-3374.