

## scil animal care company Extensive Cleaning Procedure

1. Open cover door with key.
2. From the main menu go to No. 4 for SERVICE and press enter. Then select No. 2 for DRAIN CHAMBERS and press enter to confirm.
3. Pour 5 mls of 100% Clorox bleach into the mixing chamber.
4. Pour 5 mls of 100% Clorox bleach into the red blood cell chamber.
5. From the SERVICE MENU select No. 5 for TRANSFER.
6. When the transfer is complete, select No. 1 for BACKFLUSH and repeat this step 10 TIMES.
7. Let the unit sit for a minimum of 30 minutes, but longer is better.  
**\* Please note that the longer bleach sit in lines the better chance of clog removal. OVERNIGHT is ideal!\***
8. Select No. 1 for BACKFLUSH and repeat 10 TIMES.
9. Select No. 4 for CONCENTRATED CLEANING and run this once.
10. Select No. 9 for AUTO CLEAN and run this once.

