


## Changing Reagent Packs

Starting at the MAIN MENU:

1. Press 4) SERVICE
2. Press 3) REAGENT PACK
3. Press 1) CHANGE PACK
4. CHANGE PACK? Press ENTER
5. REMOVE OLD PACK. Press any key
6. Discard old pack according to local ordinances
7. Remove red tabs from new pack's 3 bottom ports and 1 top port.

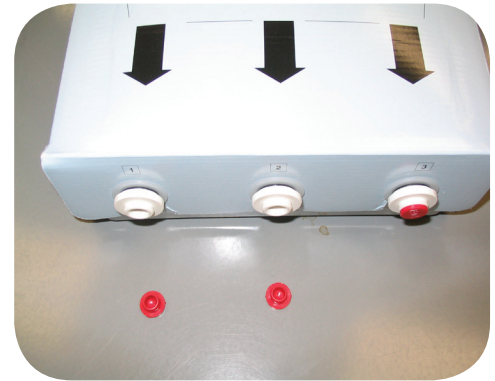

8. INSTALL NEW PACK with black arrows facing out and down. Firmly press down until little or no black line and arrows are visible.



9. Attach the waste line to the waste port on the top of the pack. Once attached, feed any excess line back into the analyzer and close the compartment lid. Please ensure that the waste line has not become kinked or trapped behind the pack or in the compartment lid.



10. Once you have installed the new pack press any key to continue. The screen will display PACK REPLACED. Press any key to continue and the analyzer will automatically start a PRIME cycle. This cycle MUST be completed to ensure adequate reagent throughout the system.

