


Instructions for software update on the EDAN i15VET

1. To guarantee power supply during the update, please connect the EDAN i15VET to the power cable.
2. Turn off the device.
3. Download the update's zip-folder (including the folders **“target”** and **“target_vet”**) from our homepage, <https://www.scilvet.de/downloads>
4. Unzip the downloaded zip-folder and transfer the folders **“target”** and **“target_vet”** to an empty USB stick, way like this f.e.:

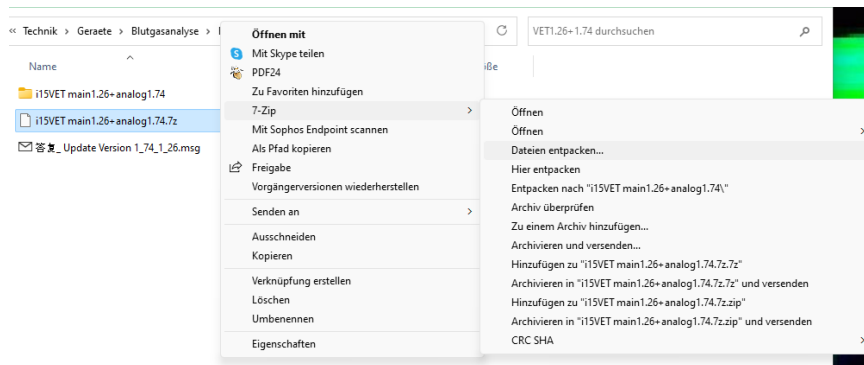

5. Insert the USB stick to a free slot of the EDAN device and turn it on
6. The update will be installed automatically (black screen).
7. As soon as the update is completed, the login mask appears in the display.
Please wait until the update is completely finished.
8. Log into the service mode:
 - a. Username: **service**
 - b. Password: **8008307573**
9. Select the **„Setup“** menu item.
10. Select the **„Diagnostics“** menu item.
11. Select the item **„Analog-Actual“** and confirm with OK.
12. The analog update launches automatically.



13. After the update is completed, the device turns off automatically.
 14. Please remove the USB stick.
 15. Turn the device on.
 16. Log into the service mode:
 - a. Username: service
 - b. Password: 8008307573
 17. Select the „**Setup**“ menu item.
 18. Select the „**Diagnostics**“ menu item.
 19. Select the item " **Analog board** " and check if the current software version **V1.74** is displayed.
 20. Select the item " **Main board** " and check if the current software version **V1.26** is displayed.
 21. If both versions are displayed correctly, the update has been installed successfully and the device can be turned off.
- If a deviating software version is shown on the device's display, please contact the technical service of scil animal care company GmbH at: **+49 (0)6204 7890 221**.