Fructosamin-VET control kit

For veterinary use only!

Control kit for quality control procedures of the quantitative veterinary in vitro determination with the Fructosamin (gold standard)-VET test kit on solo laboratory photometer.

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Kit size

16 tests/kit
1 x 1 ml

Test kit preparation: Allow single test min. 10 minutes to warm up to room temperature by placing the test into the test kit rack. Put test kit package back into refrigerator.

Summary

The Fructosamin-VET control is intended for use as a test sample of known concentration for the Fructosamin (gold standard)-VET assay. The user can compare observations with expected ranges as a means of assuring consistent performance of the Fructosamin (gold standard)-VET assay.

Preparation and Handling

Control material is lyophilized and must be reconstituted before use!

1. Open the vial very carefully, avoiding any loss of the lyophilized material.
2. Add exactly 1ml of distilled/deionized water.
Warning: Inaccurate reconstitution of the control and error in assay technique can cause erroneous results.
3. Close the vial carefully and dissolve the components completely by occasional gently swirling. Avoid the formation of foam.
4. Allow the contents to reconstitute over a period of approx. 30 minutes.
5. Analyze the control in the same way as a patient sample.

Storage and Stability

Store at 2-8°C. Control is stable until the expiration date on the label.

Stability after reconstitution:

at 2-8°C: 2 weeks

Close immediately after use. Store control tightly capped when not in use.

Waste Management

Please refer to local legal requirements.

Warnings and Precautions

Kit is for in vitro diagnostics use only. Do not ingest! Exercise the normal precautions required for handling all laboratory reagents.

All human material should be considered potentially infectious. All products derived from human blood are prepared exclusively from the blood of donors tested individually and shown to be free from HBsAg and antibodies to HCV and HIV. The testing methods applied were FDA-approved or cleared in compliance with the European Directive 98/79/EX, Annex II, List A.

However, as no testing method can rule out the potential risk of infection with absolute certainty, the material should be treated just as carefully as a patient specimen. In the event of exposure the directives of the responsible health authorities should be followed.

Warnings and Information

Set the the samplotype to "CtrlPlasma" and sex to "Control" before running the QC sample.

Compare the results with the values indicated in the lot specific value sheet included in the control kit.

If the result is not within the acceptable range repeat measurement. If result is again not within the acceptable range please contact your distributor.

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
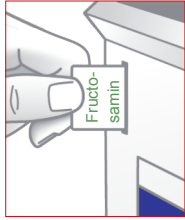
Processing of Fructosamin-VET control measurement

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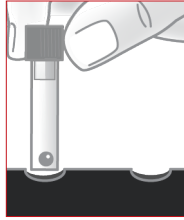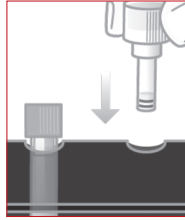
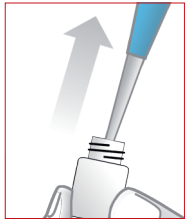
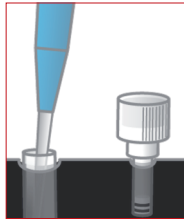
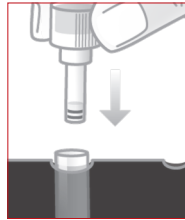
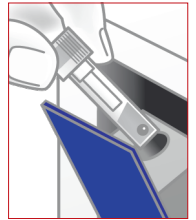
English

ATTENTION!

Allow single test min. 10 minutes to warm up to room temperature before use!

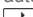
1. Prepare testsystem

- 1.1 Insert RFID-card into analyser
- 1.2 Place ERS cuvette into solo testkit rack.
- 1.3 Place ERS cap into solo testkit rack.
- 1.4 Press symbol on touch screen, required information is entered by use of the touch display of the analyser

2. For further details please refer to the according section in the solo user manual

- 2.1 Set species to: "Control"
- 2.2 Choose samplotype: Please make sure "CtrlPlasma" is selected.
- 2.3 Perform sample preparation

3. Sample preparation

- 3.1 Aspirate 20µl reconstituted control liquid with pipette
- 3.2 Pipette 20µl control liquid into ERS cuvette INTO LIQUID
- 3.3 Place ERS cap firmly on ERS cuvette
- 3.4 Place ERS cartridge into solo analyzer - start automatic sample processing by pressing the  start button

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