# Cortisol

## VET control kit

For veterinary use only!

**Intended use:**

For quality control procedures of the quantitative determination with a Cortisol VET test kit on a solo or CUBE-VET analyser

Eurolyser Diagnostica GmbH  
Bindergasse 3  
5020 Salzburg/Austria

Tel. +43 (0)662 / 43 21 00  
Fax +43 (0)662 / 43 21 00-50  
www.eurolyser.com

## English

### Order information

Order number: VT0290  
Order number: VT0291  
Order number: VT2900

### Indication

Cortisol VET test kit  
Cortisol VET test kit  
Cortisol VET control kit

### Kit size

16 tests  
6 tests  
1 x 0.5 ml (decision level)

**Test kit preparation: AFTER preparing the reagent, allow single test at least 10 min. to warm up to room temperature (20 – 25 °C) by placing the test into the test kit rack. Put test kit package back into refrigerator.**

### Summary

This Cortisol VET control kit is a valuable tool for assuring quality and precision of the cortisol determination with a Cortisol VET test kit and provides a reliable marker for monitoring the accuracy of results.

### Required Material

(not supplied with the kit)

- Pipettes
- Distilled or deionised water

### Preparation and Handling

Control material is lyophilised and must be reconstituted before use!

1. Open the vial very carefully, avoiding any loss of the lyophilised material.
2. Add exactly 0.5 ml of distilled or deionised water.
3. Let the vial rest for 30 minutes.
4. Afterwards mix by gentle swirling or inversion

**ATTENTION:** Inaccurate reconstitution of the control and error in assay technique can cause erroneous results.

### Stability and Storage

Unopened vials are stable until the expiration date stated on the label when stored at 2 – 8 °C. After reconstitution, the controls are stable for 3 days, when stored at 2 – 8 °C. Close immediately after use. Store control tightly capped when not in use

### Waste Management

Please refer to local legal requirements

### Warnings and Precautions

This control kit is for veterinary diagnostic use only!  
**DO NOT INGEST!** Exercise the normal precautions required for handling all laboratory reagents.  
As no testing method can rule out the potential risk of infection with absolute certainty, the material should be treated just as carefully as a patient specimen. In the event of exposure the directives of the responsible health authorities should be followed.

Set the species to „Control“ before running the QC material

Compare the results with the values indicated in the lot-specific value table

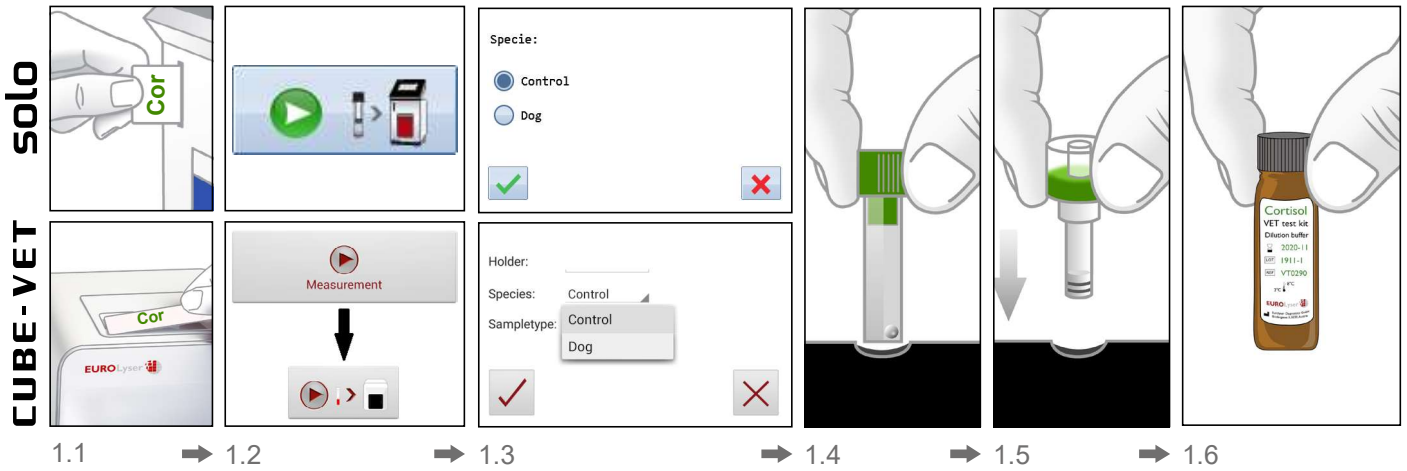
If the result is not within the acceptable range, repeat measurement. If result is again not within the acceptable range, please contact your distributor

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
## Processing of a **Cortisol** VET control measurement

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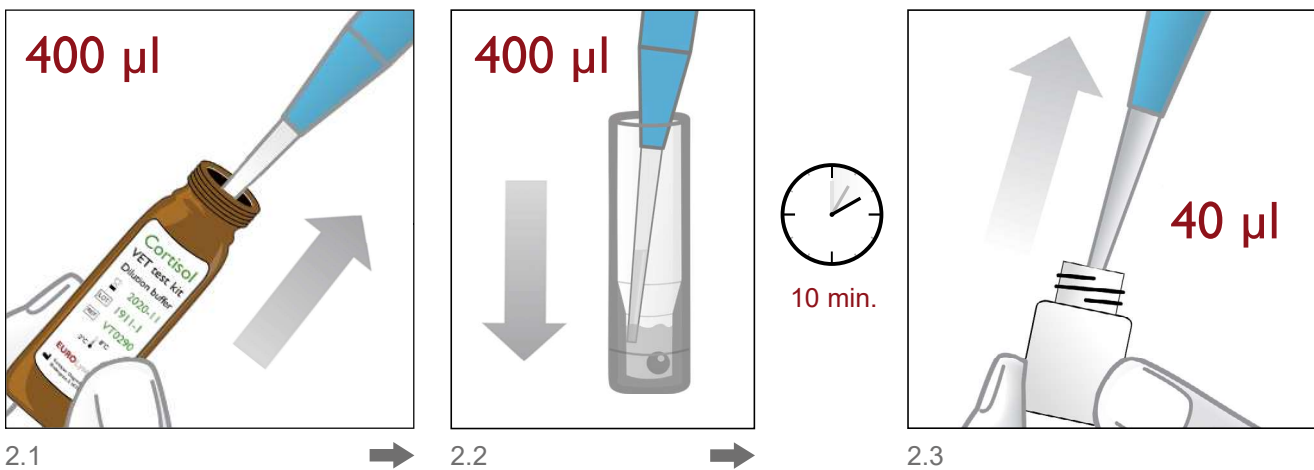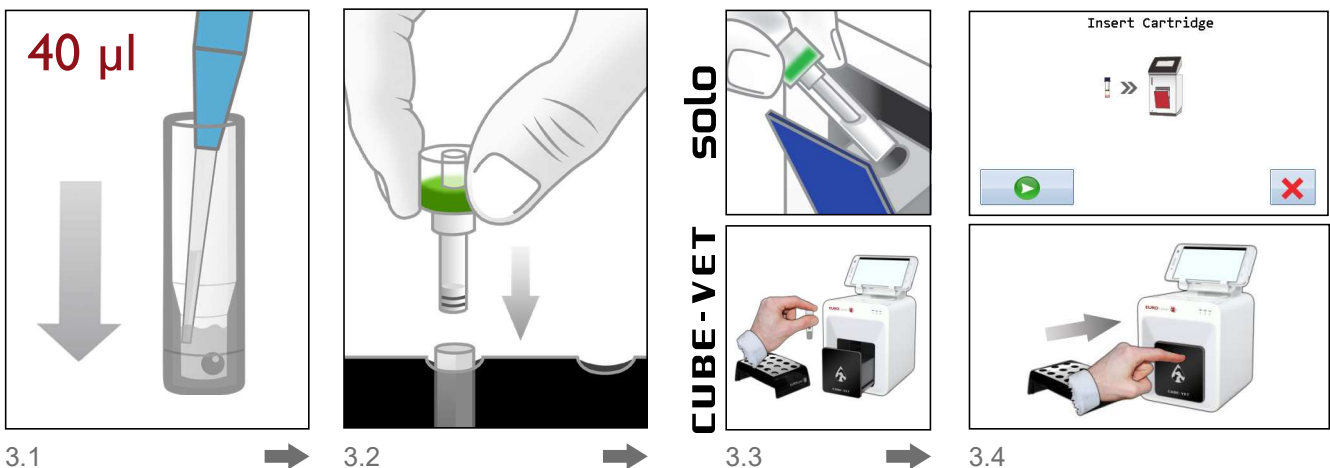
## English

### ATTENTION!

**AFTER** preparing the reagent, allow single test at least 10 minutes to warm up to room temperature (20 – 25 °C) before use!

#### 1. Preparation of test system

- 1.1 Place RFID card
- 1.2 Press „Measurement“ button, enter required information using the touchscreen
- 1.3 Set species to „Control“
- 1.4 Place ERS cuvette in test kit rack
- 1.5 Place ERS cap in test kit rack
- 1.6 Take out bottle containing dilution buffer from the test kit package

#### 2. Reagent and control preparation

- 2.1 Aspirate 400 µl from the dilution buffer bottle
- 2.2 Dispense 400 µl dilution buffer INTO THE LIQUID in the ERS cuvette

**Allow single test to warm up to room temperature for at least 10 minutes before measurement! Put test kit package back into refrigerator**

- 2.3 Aspirate 40 µl control liquid out of control vial using a pipette

#### 3. Control processing

- 3.1 Dispense 40 µl control liquid INTO THE LIQUID in the ERS cuvette
- 3.2 Apply ERS cap firmly onto ERS cuvette
- 3.3 Place ERS cartridge into laboratory photometer
- 3.4 Start automatic control processing by pressing the  start button on the solo laboratory photometer, or by closing the door of the CUBE-VET laboratory photometer.

## Wertetabelle für das Cortisol VET Control Kit Value sheet for the Cortisol VET control kit

LOT

2012-1

### Kontrollwerte | Control Values

2021-07

Target	value (min.)	value (max.)
6.0 µg/dl	3.6 µg/dl	8.4 µg/dl
166 nmol/l	99 nmol/l	232 nmol/l

Technical details subject to change without notice.

Doc-ID: e07eaa6a-baee-4139-be32-c2b2f856c22c ; latest revision: DP1.0 ; date of print: 2019-11-12

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