# Lactate VET control kit

**For veterinary use only!**

For quality control procedures of the quantitative determination with the Lactate VET test kit on a solo or CUBE-VET analyser

Eurolyser Diagnostica GmbH  
Bindergasse 3  
5020 Salzburg/Austria

Tel. +43 (0)662 / 43 21 00  
Fax +43 (0)662 / 43 21 00-50  
www.eurolyser.com

## English

|  | Order information   | Indication  | Kit size   |
|--|---|---|--|
|  | Order number: VT 0220<br>Order number: VT 0221<br>Order number: VT 2200 | Lactate VET test kit<br>Lactate VET test kit<br>Lactate VET control kit | 16 tests/pack<br>6 tests/pack<br>1 x 5 ml (decision level) |

**Test kit preparation: Allow single test at least 10 minutes to warm up to room temperature (20 - 25 °C) by placing the test into the test kit rack. Put test kit package back into refrigerator.**

### Summary

This Lactate VET control kit is a valuable tool for assuring quality and precision of the Lactate determination with the Lactate VET test kit (VT0220 / VT0221) and provides a reliable marker for monitoring the accuracy of results.

### Required Material

*(Not supplied with the kit)*

- Pipettes
- Distilled water

### Preparation and Handling

Control material is lyophilised and must be reconstituted before use!

1. Open the vial very carefully, avoiding any loss of the lyophilised material.
2. Add exactly 5 ml of distilled water.
3. Close the vial carefully and dissolve the components completely by occasional gentle swirling. Avoid the formation of foam.
4. Allow the contents to reconstitute over a period of approximately 30 minutes.
5. Analyse the control in the same way as a patient sample.

Warning: Inaccurate reconstitution of the control and error in assay technique can cause erroneous results.

### Stability and Storage

Unopened vials are stable at 2 - 8 °C until the expiration date stated on the label.

*Stability after reconstitution:*

|                       |         |
|-----------------------|---------|
| Stored at 20 - 25 °C: | 8 hours |
| Stored at 2 - 8 °C:   | 7 days  |
| Stored at -20 °C:     | 30 days |

Close immediately after use. Store control tightly capped when not in use. Warning: FREEZE ONLY ONCE!

### Waste Management

Please refer to local legal requirements

### Warnings and Precautions

This control kit is for veterinary diagnostic use only!  
**DO NOT INGEST!** Exercise the normal precautions required for handling all laboratory reagents. As no testing method can rule out the potential risk of infection with absolute certainty, the material should be treated just as carefully as a patient specimen. In the event of exposure the directives of the responsible health authorities should be followed.

Set the species to „Control“ before running the QC material

Compare the results with the values indicated in the lot-specific value table

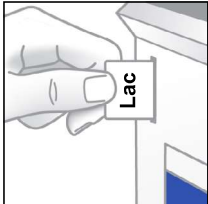
If the result is not within the acceptable range, repeat measurement. If result is again not within the acceptable range, please contact your distributor

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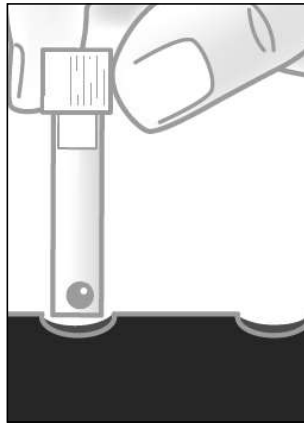
## Processing of a Lactate VET control measurement

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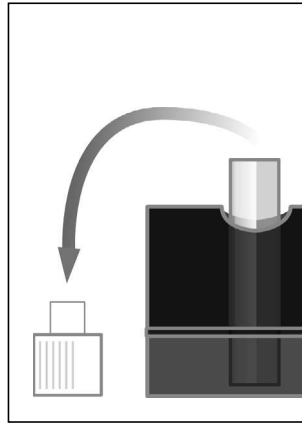
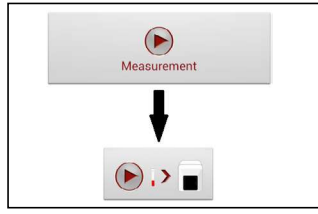
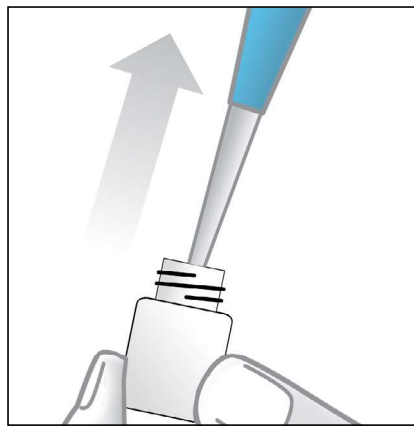
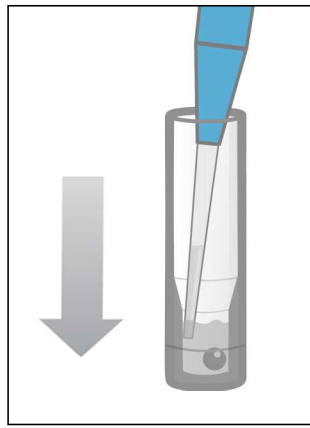
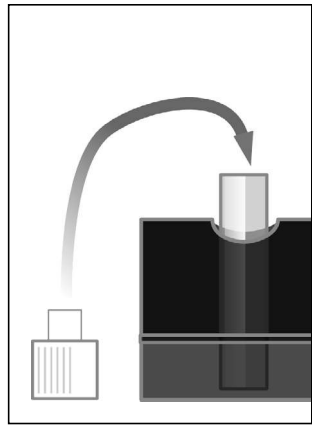
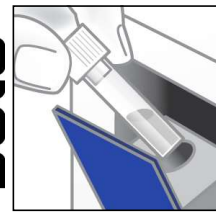
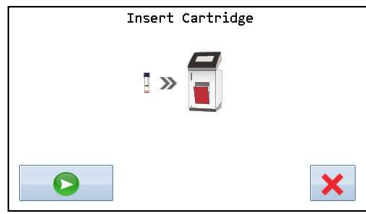
## English

### ATTENTION!

Allow single test at least 10 minutes to warm up to room temperature (20 - 25 °C) before use!

- 1. Preparation of test system**
  - 1.1 Place RFID card
  - 1.2 Place ERS cuvette in test kit rack
  - 1.3 Detach cap from ERS cuvette
  - 1.4 Press „Measurement“ button, enter required information using the touchscreen

- 2. Control preparation**

*For further details please see user manual of laboratory photometer*

  - 2.1 Set species to „Control“
  - 2.2 Aspirate 20 µl control liquid using a pipette

- 3. Control processing**
  - 3.1 Dispense 20 µl control liquid INTO THE LIQUID in the ERS cuvette
  - 3.2 Apply cap firmly onto ERS cuvette
  - 3.3 Place ERS cartridge into laboratory photometer
  - 3.4 Start automatic control processing by pressing the  start button on the solo laboratory photometer, or by closing the door of the CUBE-VET laboratory photometer.

## Wertetabelle für das Lactate VET Control Kit Value sheet for the Lactate VET control kit

LOT

2410-1

2025-09

### Kontrollwerte | Control Values

| Target      | value (min) | value (max) |
|-------------|-------------|-------------|
| 1.42 mmol/l | 1.09 mmol/l | 1.75 mmol/l |

Technical details subject to change without notice.

Doc-ID: 9151c3e5-7273-434e-b74b-d2fc7bab171b ; latest revision: DP 2.0 ; date of print: 2018-06-13

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