



Instructions for installing the QuickVet® - software

1. Open the URL in a web browser and save the application file (exe.Datei) on your drive.
2. Click on the „Run“ Button

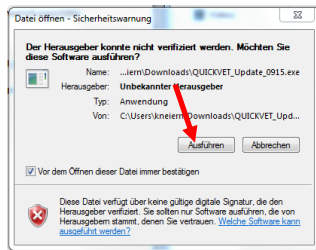


3. Navigate to the USB drive and click on "Extract".

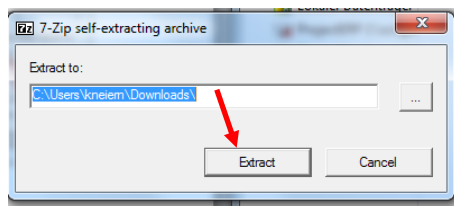

4. Enter the password (see letter) and click „OK“.



5. Turn the QuickVet on.
6. Insert the USB stick into the USB port on the rear panel
7. Select „Settings“ from the menu.
8. Select the tab „**Software V.**“ .
9.  Press to open the menu
10. Press **Upgrade**
11. Prompt: Upgrade to newer version? Press **Yes** Press Yes to confirm the update
12. The update will start automatically
13. After completing the update the device will restart automatically.



14. Select „ **Settings**“ from the Menu.
15. Select “Client”, check „Settings“ and change if needed:
16. Patient –ID required ja
 Sample-Nr required nein
 Calibrations reminder nein
 Fibrinogen Unit g/l
 Sample supply prompt: Once
 Done Done
17. Turn off the device