

scil Vet abc

Simple - Fast - Reliable

Analytical Device for the Determination of
Haematological Parameters in Whole Blood!



scil Vet abc

Analytical Device for the Determination of Haematological Parameters in Whole Blood

The scil Vet abc is simple to operate and produces a complete three part differential blood count in only 90 seconds. When the measurement is started, 12 µl of EDTA whole blood are automatically aspirated - no sample preparation is necessary. The results are automatically transferred to your practice software program.

The operator-friendly reagent pack concept renders daily maintenance procedures unnecessary. The proven impedance technology delivers absolutely reliable results in healthy animals as well as of course in sick ones.



Handling:



Step 1:
Insert Smart Card



Step 2:
Add EDTA sample
and press start



Step 3:
Read results in 90
seconds

Technical Data:

Species:	Dog, Cat, Horse, Rat, Pig, Cow, Rabbit, Sheep, Mouse, Monkey, Ferret
Parameters:	WBC, RBC, HGB, HCT, MCV, MCH, MCHC, RDW, RBC, PLT, MPV
	Additional Parameters for Dog, Cat, Horse, Ferret: Lymphocyte # + %, Monocyte, # + %, Granulocyte # + %, Eosinophil %
Theory of Operation:	RBC, WBC and PLTS by electrical impedance, Hemoglobin by spectrophotometry
Computer Interface:	Serial, RS 232
Size:	330 x 360 x 440 mm
Weight:	31 lbs. / ca. 14 kg

Distributed by:



scil animal care company GmbH
Dina-Weissmann-Allee 6
68519 Viernheim
Germany
Tel.: +49 (0) 6204 78 90 0
Fax: +49 (0) 6204 78 90 200
Email: info-de@scilvet.com
www.scilvet.com