Fibrinogen VET control kit

Français

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English

For veterinary use only!

Intended use: For quality control procedures of the quantitative determination with the Fibrinogen VET test kit on a solo or CUBE-VET analyser.

Order information	Indication	Kit size
Order number: VT0120	Fibrinogen test kit	16 tests
Order number: VT0121	Fibrinogen test kit	6 tests
Order number: VT1200	Fibrinogen control kit	1 x 1 ml (dec. level)

 Test kit preparation: Allow single test at least 10 minutes to warm up to room temperature (20 – 25 °C) by placing the test into the test kit rack. Put test kit package back into refrigerator.

Summary

The use of this Fibrinogen VET control kit serves to reliably assess the quality and precision of the fibrinogen determination with a Fibrinogen VET test kit (VT0120, VT0121) and therefore enables the monitoring of the accuracy of the test results.

Required Material

(not supplied with the kit)

- Pipettes
- Distilled water

Preparation and Handling

Open the vial carefully and reconstitute the contents in 1 ml of distilled water by carefully rotating the vial (avoid frothing).

Allow reconstituted controls to stand for 10 minutes at room temperature before use.

Stability and Storage

Unopened vials are stable until the expiration date stated on the label when stored at 2 – 8 °C.

Stability after reconstitution:

at 20 – 25 °C:	4 hours
at 2 – 8 °C:	8 hours
at -20 °C:	8 weeks

The vials can only be frozen once. Upon storage, caps should be screwed tightly.

Warnings and Precautions

This control kit is for veterinary diagnostic use only!

-  The control components derived from material of human origin were found to be non-reactive for hepatitis B surface antigen (HBsAg), hepatitis C (HCV), HIV-1 and HIV-2. Nevertheless, the control must be regarded as potentially infectious. It must be handled with the necessary care and in accordance with the safety regulations.
-  Set the species to “*Control*” and the sample type to “*Ctrl.Plasma/Control Plasma*” before running the QC material
-  Compare the results with the values indicated in the lot-specific value table.
-  If the result is not within the acceptable range, repeat measurement. If result is again not within the acceptable range, please contact your distributor.

Waste Management

Please refer to local legal requirements

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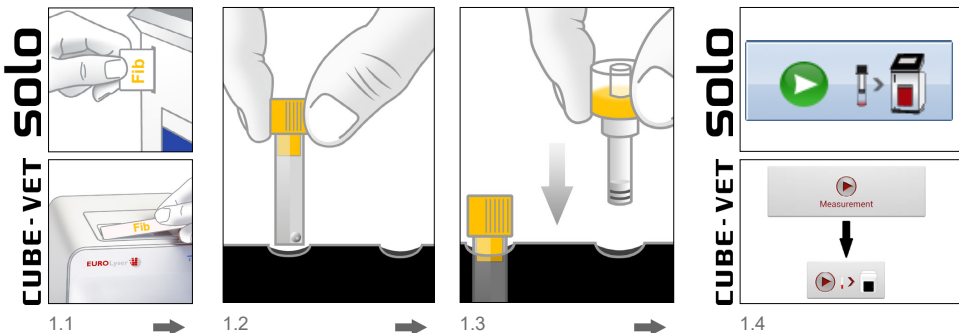
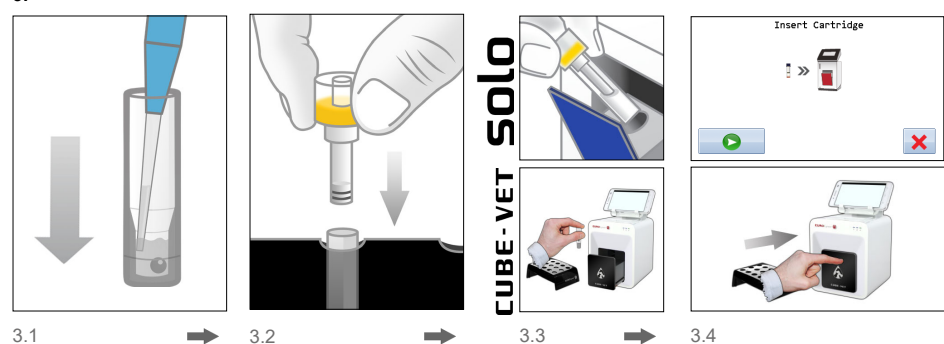
English

ATTENTION!

Allow single test at least 10 minutes to warm up to room temperature (20 - 25 °C) before use!

1. Preparation of test system

- 1.1 Place RFID card
- 1.2 Place ERS cuvette in test kit rack
- 1.3 Place ERS cap in test kit rack
- 1.4 Press „Measurement“ button, enter required information using the touch screen

2. Control preparation

For further details please see user manual of laboratory photometer

- 2.1 Set species to „Control“
- 2.2 Set sampletype to „CtrlPlasma/Control Plasma“
- 2.3 Aspirate 5 µl control liquid using a pipette.

3. Control processing

- 3.1 Dispense 5 µl control liquid INTO THE LIQUID in ERS cuvette.
- 3.2 Apply ERS cap firmly onto ERS cuvette.
- 3.3 Place ERS cartridge into laboratory photometer.
- 3.4 Start automatic control processing by pressing the start button on the solo laboratory photometer or by closing the door of the CUBE-VET laboratory photometer

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