Fructosamine VET control kit

Français

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English

For veterinary use only!

Intended use: For quality control procedures of the quantitative determination with the Fructosamine (gold standard) VET test kit on a solo or CUBE-VET analyser.

Order information	Indication	Kit size
Order number: VT0110	Fructosamine test kit	16 tests
Order number: VT0111	Fructosamine test kit	6 tests
Order number: VT1100	Fructosamine control kit	1 x 1 ml (dec. level)

 **Test kit preparation: Allow single test at least 10 minutes to warm up to room temperature (20 – 25 °C) by placing the test into the test kit rack. Put test kit package back into refrigerator.**

Summary

The use of this Fructosamine VET control kit serves to reliably assess the quality and precision of fructosamine determination with a Fructosamine (gold standard) VET test kit (VT0110, VT0111) and therefore enables the monitoring of the accuracy of the test results.

Required Material

(not supplied with the kit)

- Pipettes
- Distilled water

Preparation and Handling

Control material is lyophilized and must be reconstituted before use!

1. Open the vial very carefully, avoiding any loss of the lyophilized material.
2. Add exactly 1 ml of distilled/deionized water.
Warning: Inaccurate reconstitution of the control and error in assay technique can cause erroneous results.
3. Close the vial carefully and dissolve the components completely by occasional gently swirling. Avoid the formation of foam.
4. Allow the contents to reconstitute over 30 minutes before.
5. Analyse the control in the same way as a patient sample.

Stability and Storage

Unopened vials are stable until the expiration date stated on the label when stored at 2 – 8 °C.

Stability after reconstitution:
at 2 – 8 °C: 2 weeks

Close immediately after use. Store control tightly capped when not in use.

Warnings and Precautions

This control kit is for veterinary diagnostic use only!

DO NOT INGEST! Exercise the normal precautions required for handling all laboratory reagents.

All products derived from human blood are prepared exclusively from the blood of donors tested individually and shown to be free from HBsAg and antibodies to HCV and HIV-1 and HIV-2. The testing methods applied were FDA-approved or cleared in compliance with the European Directive 98/79/EX, Annex II, List A. However, as no testing method can rule out the potential risk of infection with absolute certainty, the material should be treated just as carefully as a patient specimen. In the event of exposure the directives of the responsible health authorities should be followed.

 Set the species to „*Control*“ and the sample type to „*CtrlPlasma*“/„*Control Plasma*“ before running the QC material.

 Compare the results with the values indicated in the lot-specific value table.

 If the result is not within the acceptable range, repeat measurement. If result is again not within the acceptable range, please contact your distributor.

Waste Management

Please refer to local legal requirements.

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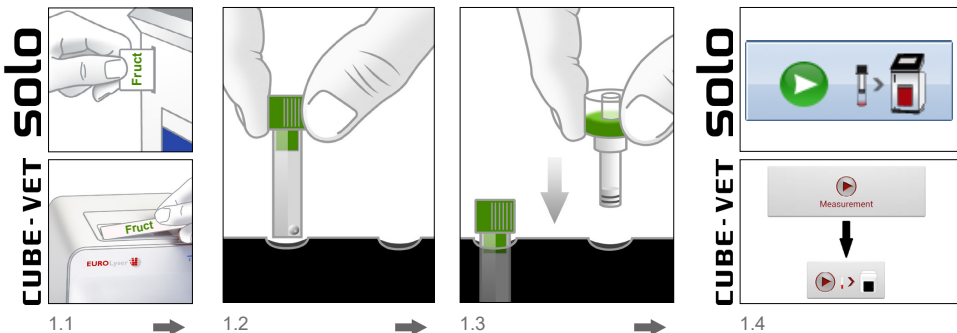
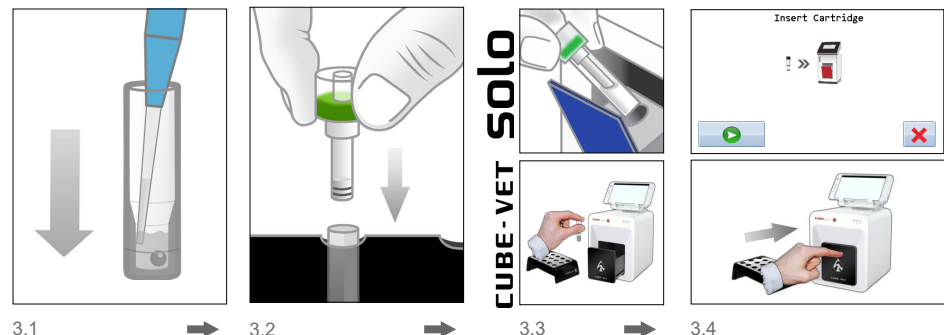
English

ATTENTION!

Allow single test at least 10 minutes to warm up to room temperature (20 - 25 °C) before use!

1. Preparation of test system

- 1.1 Place RFID card
- 1.2 Place ERS cuvette in test kit rack
- 1.3 Place ERS cap in test kit rack
- 1.4 Press „Measurement“ button, enter required information using the touch screen

2. Control preparation

For further details please see user manual of laboratory photometer

- 2.1 Set species to „Control“
- 2.2 Set sample type to „CtrlPlasma“, „Control Plasma“
- 2.3 Aspirate 20 µl control liquid using a pipette

3. Control processing

- 3.1 Dispense 20 µl control liquid INTO THE LIQUID IN ERS cuvette.
- 3.2 Apply ERS cap firmly onto ERS cuvette.
- 3.3 Place ERS cartridge into laboratory photometer.
- 3.4 Start automatic control processing by pressing the start button on the solo laboratory photometer or by closing the door of the CUBE-VET laboratory photometer

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