GLDH VET control kit

Français

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English

For veterinary use only!

Intended use: For quality control procedures of the quantitative determination with the GLDH VET test kit on a solo or CUBE-VET analyser.

Order information	Indication	Kit size
Order number: VT0200	GLDH test kit	16 tests
Order number: VT0201	GLDH test kit	6 tests
Order number: VT2000	GLDH control kit	1 x 5 ml (dec. level)

 **Test kit preparation: Allow single test at least 10 minutes to warm up to room temperature (20 – 25 °C) by placing the test into the test kit rack. Put test kit package back into refrigerator.**

Summary

The use of this GLDH VET control kit serves to reliably assess the quality and precision of GLDH determination with a GLDH VET test kit (VT0200, VT0201) and therefore enables the monitoring of the accuracy of the test results.

Required Material

(not supplied with the kit)
- Pipettes
- Distilled water

Preparation and Handling

Control material is lyophilized and must be reconstituted before use!

- Open the vial very carefully, avoiding any loss of the lyophilized material.
- Add exactly 5 ml of distilled/deionized water.
- Close the vial carefully and dissolve the components completely by occasional gently swirling. Avoid the formation of foam.
- Allow the contents to reconstitute over 30 minutes before.
- Analyse the control in the same way as a patient sample.

Warning: Inaccurate reconstitution of the control and error in assay technique can cause erroneous results.

Stability and Storage

Unopened vials are stable until the expiration date stated on the label when stored at 2 - 8 °C.

Stability after reconstitution:

at 20 - 25 °C: 8 hours
at 2 - 8 °C: 7 days
at -20 °C: 30 days

Close immediately after use. Store control tightly capped when not in use. ATTENTION: FREEZE ONLY ONCE!

Warnings and Precautions

This control kit is for veterinary diagnostic use only!

DO NOT INGEST! Exercise the normal precautions required for handling all laboratory reagents.

As no testing method can rule out the potential risk of infection with absolute certainty, the material should be treated just as carefully as a patient specimen. In the event of exposure the directives of the responsible health authorities should be followed.

-  Set the species to „Control” before running the QC material.
-  Compare the results with the values indicated in the lot-specific value table.
-  If the result is not within the acceptable range, repeat measurement. If result is again not within the acceptable range, please contact your distributor.

Waste Management

Please refer to local legal requirements.

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