Phenobarbital VET control kit

Français

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English

For veterinary use only!

Intended use: For quality control procedures of the quantitative determination with the Phenobarbital VET test kit on a solo or CUBE-VET analyser.

Order information	Indication	Kit size
Order number: VT0211 Order number: VT2100	Phenobarbital test kit Phenobarbital control kit	6 tests 1 x 5 ml (dec. level)

⚠ **Test kit preparation:** Allow single test at least 10 minutes to warm up to room temperature (20 – 25 °C) by placing the test into the test kit rack. Put test kit package back into refrigerator.

Summary

The use of this Phenobarbital VET control kit serves to reliably assess the quality and precision of the phenobarbital determination with an Phenobarbital VET test kit (VT0211) and therefore enables the monitoring of the accuracy of the test results.

Required Material

(not supplied with the kit)

- Pipettes

Preparation and Handling

The control is liquid and ready to use. Thoroughly mix the contents of the vial before each use by gently inverting for several minutes. Open the vial and transfer the required quantity of control into a clean sample cup. Use sample cup only for one measurement circle and use new sample cups further on to avoid contamination of the control liquid. Replace the cap immediately and store the opened vial at 2 – 8 °C. Always use disposable pipette tips!

Stability and Storage

Unopened vials are stable at 2 – 8 °C until the expiration date stated on the label. Once opened, vials of control are stable up to 30 days when stored tightly capped at 2 – 8 °C. Bacterial contamination produces an increase in turbidity and/or a characteristic odor. Discard vial if evidence of microbial contamination is observed. DO NOT FREEZE!

Warnings and Precautions

This control kit is for veterinary diagnostic use only!

DO NOT INGEST! Exercise the normal precautions required for handling all laboratory reagents.

- ⚠ Set the species to „*Control*” before running the QC material.
- ⚠ Compare the results with the values indicated in the lot-specific value table.
- ⚠ If the result is not within the acceptable range, repeat measurement. If result is again not within the acceptable range, please contact your distributor.

Waste Management

Please refer to local legal requirements.

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English

ATTENTION!

Allow single test at least 10 minutes to warm up to room temperature (20 - 25 °C) before use!

1. Preparation of test system
 - 1.1 Place RFID card
 - 1.2 Place ERS cuvette in test kit rack
 - 1.3 Place ERS cap in test kit rack
 - 1.4 Press „Measurement“ button, enter required information using the touch screen

2. Control preparation

For further details please see user manual of laboratory photometer

 - 2.1 Set species to „Control“
 - 2.2 Aspirate 20 µl control liquid using a pipette

3. Control processing
 - 3.1 Dispense 20 µl control liquid INTO THE LIQUID in ERS cuvette.
 - 3.2 Apply ERS cap firmly onto ERS cuvette.
 - 3.3 Place ERS cartridge into laboratory photometer.
 - 3.4 Start automatic control processing by pressing the start button on the solo laboratory photometer or by closing the door of the CUBE-VET laboratory photometer

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