## SAA VET control kit

### Français

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### English

For veterinary use only!

**Intended use:** For quality control procedures of the quantitative determination with the SAA VET test kit on a solo or CUBE-VET analyser.

| Order information    | Indication      | Kit size              |
|----------------------|-----------------|-----------------------|
| Order number: VT0170 | SAA test kit    | 6 tests               |
| Order number: VT0171 | SAA test kit    | 16 tests              |
| Order number: VT1700 | SAA control kit | 1 x 2 ml (dec. level) |

 **Test kit preparation: Allow single test at least 10 minutes to warm up to room temperature (20 – 25 °C) by placing the test into the test kit rack. Put test kit package back into refrigerator.**

### Summary

The use of this SAA VET control kit serves to reliably assess the quality and precision of the SAA determination with an SAA VET test kit (VT0170, VT0171) and therefore enables the monitoring of the accuracy of the test results.

### Required Material

(not supplied with the kit)  
- Pipettes  
- Distilled water

### Preparation and Handling

Control material is lyophilized and must be reconstituted before use!

- Open the vial very carefully, avoiding any loss of the lyophilized material.
- Add exactly 2 ml of distilled/deionized water.  
Warning: Inaccurate reconstitution of the control and error in assay technique can cause erroneous results.
- Close the vial carefully and dissolve the components completely by occasional gently swirling. Avoid the formation of foam.
- Allow the contents to reconstitute over 15 minutes before.
- Analyse the control in the same way as a patient sample.

### Stability and Storage

Unopened vials are stable until the expiration date stated on the label when stored at 2 - 8 °C.

**Stability after reconstitution:**  
at 2 - 10 °C: 2 weeks

Close immediately after use. Store control tightly capped when not in use. Bacterial contamination produces an increase in turbidity and/or a characteristic odor. Discard vial if evidence of microbial contamination is observed.

### Warnings and Precautions

This control kit is for veterinary diagnostic use only!

**DO NOT INGEST!** Exercise the normal precautions required for handling all laboratory reagents. The control components derived from material of human origin have been found to be non-reactive with respect to hepatitis B surface antigen (HBsAg), hepatitis C (HCV), HIV-1 and HIV-2. However, it is impossible to completely rule out the possibility that these viruses or other pathogens may be present. Therefore when using these reagents, handle them with same degree of caution as patient samples.

This control must be handled in accordance with recommendations from Centers for Disease Control/National Institutes of Health manual, "Biosafety in microbiological and biomedical laboratories" (1999).

-  Set the species to „*Control*” before running the QC material.
-  Compare the results with the values indicated in the lot-specific value table.
-  If the result is not within the acceptable range, repeat measurement. If result is again not within the acceptable range, please contact your distributor.

### Waste Management

Please refer to local legal requirements.

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