Cortisol VET control kit

Français

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English

For veterinary use only!

Intended use: For quality control procedures of the quantitative determination with the Cortisol VET test kit on a solo or CUBE-VET analyser.

Order information	Indication	Kit size
Order number: VT0290	Cortisol test kit	16 tests
Order number: VT0291	Cortisol test kit	6 tests
Order number: VT2900	Cortisol control kit	1 x 2 ml (dec. level)

⚠ **Test kit preparation:** Allow single test at least 10 minutes to warm up to room temperature (20 – 25 °C) by placing the test into the test kit rack. Put test kit package back into refrigerator.

Summary

The use of this Cortisol VET control kit serves to reliably assess the quality and precision of the cortisol determination with a Cortisol VET test kit (VT0290, VT0291) and therefore enables the monitoring of the accuracy of the test results.

Preparation and Handling

The control is liquid and ready to use.

Stability and Storage

Unopened vials are stable until the expiration date stated on the label when stored at 2 – 8 °C. Once opened, vials are stable for 5 days, when stored tightly capped at 2 – 8 °C.

Warnings and Precautions

This control kit is for veterinary diagnostic use only!

DO NOT INGEST! Exercise the normal precautions required for handling all laboratory reagents.

⚠ This product contains a substance with animal origin. Hazardous properties cannot be excluded. No dangers expected for the user if used according to directions. Donors have been tested and are found to be clinically healthy. Even though tests on infectious diseases ended negative, this product should be handled as potentially infectious biomaterial.

⚠ Set the species to „*Control*“ before running the QC material.

⚠ Compare the results with the values indicated in the lot-specific value table.

⚠ If the result is not within the acceptable range, repeat measurement. If result is again not within the acceptable range, please contact your distributor.

Waste Management

Please refer to local legal requirements

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English

ATTENTION!

Allow single test at least 10 minutes to warm up to room temperature (20 - 25 °C) before use!

1. Preparation of test system

1.1 Place RFID card

1.2 Press „Measurement“ button, enter required information using the touch screen

1.3 Set species to „Control“

1.4 Place ERS cuvette in test kit rack

1.5 Place ERS cap in test kit rack

1.6 Take out bottle containing dilution buffer from the test kit package

2. Reagent and control preparation

2.1 Aspirate 400 µl from the dilution buffer bottle

2.2 Dispense 400 µl dilution buffer INTO THE LIQUID in the ERS cuvette

2.3 Dispense some drops of control liquid into serum cup and aspirate 40 µl using the pipette.

3. Control processing

3.1 Dispense 40 µl control liquid INTO THE LIQUID in ERS cuvette.

3.2 Apply ERS cap firmly onto ERS cuvette.

3.3 Place ERS cartridge into laboratory photometer.

3.4 Start automatic control processing by pressing the start button on the solo laboratory photometer or by closing the door of the CUBE-VET laboratory photometer

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