## canine Progesterone VET control kit

### Français

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### English

For veterinary use only!

**Intended use:** For quality control procedures of the quantitative determination with the canine Progesterone VET test kit on a solo or CUBE-VET analyser.

| Order information    | Indication               | Kit size              |
|----------------------|--------------------------|-----------------------|
| Order number: VT0230 | canine Prog. test kit    | 6 tests               |
| Order number: VT0231 | canine Prog. test kit    | 16 tests              |
| Order number: VT2300 | canine Prog. control kit | 1 x 2 ml (dec. level) |

**Test kit preparation:** Allow single test at least 10 minutes to warm up to room temperature (20 – 25 °C) by placing the test into the test kit rack. Put test kit package back into refrigerator.

### Summary

The use of this canine Progesterone VET control kit serves to reliably assess the quality and precision of the progesterone determination with a canine Progesterone VET test kit (VT0230, VT0231) and therefore enables the monitoring of the accuracy of the test results.

### Preparation and Handling

The control is liquid and ready to use.

### Stability and Storage

Unopened vials are stable until the expiration date stated on the label when stored at 2 - 8 °C. Once opened, vials are stable for up to 30 days, when stored tightly capped at 2 – 8 °C.

### Warnings and Precautions

This control kit is for veterinary diagnostic use only!

DO NOT INGEST! Exercise the normal precautions required for handling all laboratory reagents.

This product contains a substance with animal origin. Hazardous properties cannot be excluded. No dangers expected for the user if used according to directions. Donors have been tested and are found to be clinically healthy. Even though tests on infectious diseases ended negative, this product should be handled as potentially infectious biomaterial.

Set the species to „*Control*“ before running the QC material.

Compare the results with the values indicated in the lot-specific value table.

If the result is not within the acceptable range, repeat measurement. If result is again not within the acceptable range, please contact your distributor.

### Waste Management

Please refer to local legal requirements

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